


# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

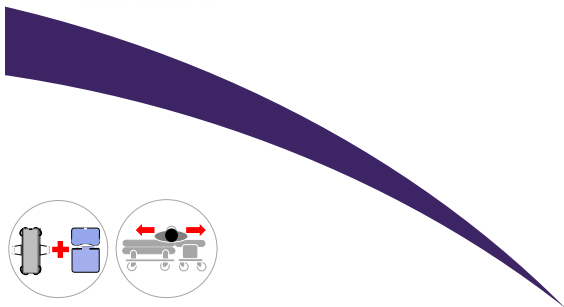

(DK) Forordning MDR (EU) 2017/745 for medicinsk udstyr klasse 1  
(UK) Regulation MDR (EU) 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic  
Human Handling Concept



## Quick guide

(DK) **Master Turner Duo** - til madrascovers  
(UK) **Master Turner Duo** - for mattress covers

10\_20230623 DK/UK



## The Original Master Turner System

Master Turner Duo - L



Vare nr./  
Item no.

Master Turner Duo Inco - XL



Navn- Størrelse/  
Name - Size

UDI-DI

<input type="radio"/>	MC 001-1700	49207	Master Turner Duo Bund/Bottom - L	5744001970539
<input type="radio"/>	MC 001-1701	49206	Master Turner Duo Top - L	5744001970546
<input type="radio"/>	MC 001-1705	61346	Master Turner Duo Top - XL	5744001970553
<input type="radio"/>	MC 001-1706	61347	Master Turner Duo Bund/Bottom - XL	5744001970560
<input type="radio"/>	MC 001-1710	106548	Master Turner Duo Air Bund/Bottom - L	5744001970577
<input type="radio"/>	MC 001-1711	106549	Master Turner Duo Air Top - L	5744001970584
<input type="radio"/>	MC 001-2500	49873	Master Turner Duo Inco Bund/Bottom - L	5744001971031
<input type="radio"/>	MC 001-2501	49872	Master Turner Duo Inco Top - L	5744001971048
<input type="radio"/>	MC 001-2502	53648	Master Turner Duo Inco Top - XL	5744001971055
<input type="radio"/>	MC 001-2503	53649	Master Turner Duo Inco Bund/Bottom - XL	5744001971062
<input type="radio"/>	MC 001-1949	Tilbehør/ Accessories	Vaskestropper - Duo Top (4 stk) / Washing straps - Duo Top (4 pcs)	5744001970911

### Eftersyn:

Produktet skal kontrolleres jævnlig og gerne inden hver brug. Kontroller, at der ikke er skader på stof, stropper, sømme mm. Har produktet høj friktion eller på anden vis er i stykker, skal produktet kasseres.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



(DK) Efterlad aldrig bruger på produktet på sengekanten.

(UK) Never leave the user on the product when lying on the edge of the bed.



(DK) Efterlad aldrig produktet liggende på gulvet.

(UK) Never leave the product on the floor.




(DK) Efterlad aldrig en bruger liggende på produktet, mens dette er koblet på en lift.

(UK) Never leave the user lying on the product whilst connected to a hoist.



(DK) Sørg altid for, at brugeren ikke hænger fast i stropperne.

(UK) Always make sure that the user is not caught in the straps.

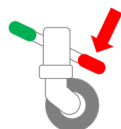

(DK) Sørg for, at alle relevante bremsere er låst, inden forflytning foretages.

(UK) Make sure that all relevant brakes are activated before transferring the user.



(DK) Må ikke bruges som liftsejl – kan med fordel kombineres med lift til fx vendinger i seng.

(UK) Do not use as a sling – can be combined with a hoist for repositioning in bed.

(DK) For mere information, brugervejledning, videoer mm., se  
(UK) For more information, User Manual, videos etc. visit


[www.master-care.dk](http://www.master-care.dk)

Developed &  
Manufactured by:




master-care.dk

# The Original Master Turner

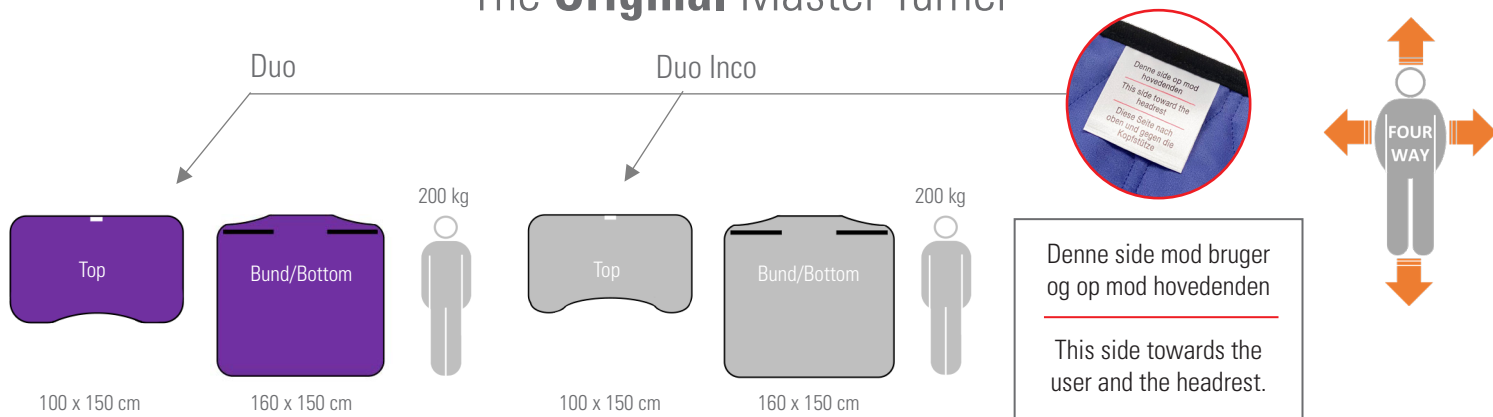

## (DK) Kombinationsmuligheder / (UK) Combinations

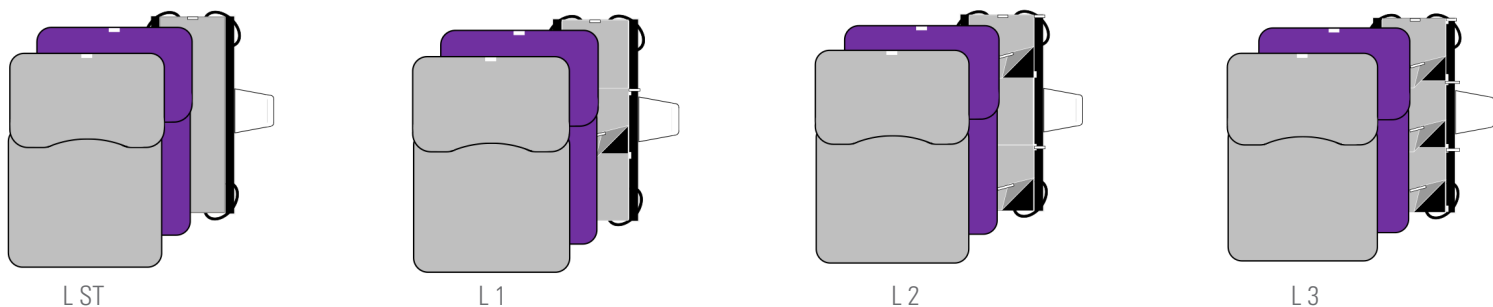

## (DK) Anvendelsesmuligheder / (UK) Applications



- (DK) Produkter med burrebånd må max. vaskes ved 60°C.  
(UK) Products with hook and loop tape must be washed at max. 60°C.
- (DK) Skånsom vask og tørretumbling forlænger holdbarheden væsentligt.  
(UK) Gentle washing and dry tumbling significantly prolongs the durability.
- (DK) Anvend aldrig skyllemiddel, da dette forringer glideegenskaberne væsentligt.  
(UK) Never use fabric softener as this will reduce the sliding properties significantly.
- (DK) Luk altid burrebånd inden vask.  
(UK) Always combine the hook and loop before washing.



Find us on:



master care A/S, Sofienlystvej 3, DK-8340 Malling, Danmark  
Tel. +45 86 93 85 85, E-mail: info@master-care.dk

(DK) Vi tager forbehold for evt. trykfejl og mangler.  
(UK) Reservations are made for printing errors.