

MADE IN EUROPE




This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

02\_20240610\_UK

## Rescue Master

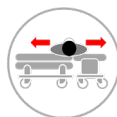

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-1110	123663	Rescue Master	5744001970218

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic Human Handling Concept



Developed & Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

## **MDR (EU) 2017/745:**

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The product is used for moving people who have fallen on the floor and need help to get up again. The product is placed under the person who can then be slid over to a more suitable area for further transfer or slid over onto a lifting device. The product can be used to reduce the risk of shear injuries.

## **Proper use of the product:**

Rescue Master is used, e.g. with users who have fallen and where it is not possible to help the user up on their own or e.g., using a hoist. Rescue Master is placed under the user. The helpers can now slide the user onto a lifting cushion or to a place with better space by pulling in the top layer of the Rescue Master.

## **Contraindications:**

If there is doubt as to whether the user has sustained a fracture, the transfer should not be carried out until this has been cleared by a relevant healthcare professional.

## **Warning:**

Due to the product's sliding properties the product should either be removed from under the user when the latter is lifted from the floor

using the lifting cushion or be fastened so that the user does not slide of the lifting cushion if it tilts.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried. Products without nylon sliding material have a much longer life span.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - Rescue Master

## Using Rescue Master combined with a lifting cushion:



1. Turn the user slightly to the side and place the folded Rescue Master under the user.



2. Turn the user slightly to the opposite side and unfold the Rescue Master.



3. Now place the lifting cushion next to the user and slide the user on top by pulling the black straps/ handles until the user is positioned on top of the lifting device.



4. Remove the Rescue Master by folding it inside it self and pull.



5. Change pull direction if resistance is met.

## Rescue Master as a sliding piece:



6. If the user has fallen where there is little space, the Rescue Master can be used as a sliding piece, to move the user to a more suitable area.



7. Fold the Rescue Master under the user and pull the user to a more suitable area for transfer or treatment.



8. Also if a sliding piece is needed during transfer between two beds, the Rescue Master can be used. Master Glide Thor can be used as a "bridge" between the beds.

# User Manual - Rescue Master

## Technical Specifications

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Max. user height:	Max. user weight:	Product contents:	Product weight:
MC 001-1110 Rescue Master	L 153 x B 148 cm	Not relevant	250 kg	7% Polyester, 84% Polyamide, 9% Silicone	0,5 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



### User Weight:

Recommended max. user weight: 250 kg



### Washing:

This product can be washed in a washing machine at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



**Dry Tumbling:** This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)



Find us on:

