

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

14_20240104_UK

Master Turner - S


The **Original** Master Turner System



MC 001-1712
MC 001-1912



MC 001-2517

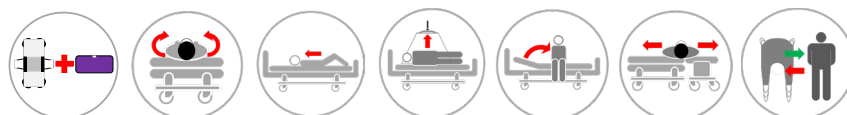

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 001-1712	50302	Master Turner Classic - S	5744001970591
<input type="radio"/> MC 001-2517	53666	Master Turner Classic Inco - S	5744001971079
<input type="radio"/> MC 001-1912	33248	Master Turner Terry Cloth - S	5744001973240

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic Human Handling Concept



Developed & Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The intended use for the product is to reduce friction between the user and the mattress to make it easier for the helper and gentler for the user whilst turning, moving up/downwards, and moving to sitting position on the edge of the bed. This product should always be used in combination with a Mattress Cover from master care a/s.

Correct use of the product:

The combination of the Master Turner and the Mattress Cover from master care A/S constitutes a turning system, which can be used for users with a decrease in function where transfers in bed and between beds can be difficult.

Risk assessment:

Always make a risk assessment and a functional assessment of the

user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Master Turner - S

Table of Contents

Mounting a Mattress Cover and Master Turner – S:	page 2
Getting into bed independently with Master Turner – S:	page 2
Turning to side lying position: Independently, Manually with one helper or With a ceiling hoist:	page 2
Prevent sliding down in bed:	page 3
Technical Specifications	page 4



User Manual - Master Turner - S

Mounting a Mattress Cover - S and Master Turner – S:



1. Mount the Mattress Cover - S on the mattress.



2. Place the Master Turner - S with the nylon side downwards and the purple side up.



3. The Master Turner is ready to use.

Getting into bed independently with Master Turner – S:



4. Place the Master Turner – S across the bed. The handles are facing towards the head end and the foot end. The user sits down on the edge of the bed on the middle of the Turner but with the pelvis well into the bed.



5. Master Turner can be folded around the user's pelvis. Place one hand on the bed rail and turn into the bed. The user lifts up the legs when needed.



6. Straighten out the Master Turner if it was wrapped around the user.

Turning to side lying position: Independently, Manually with one helper or With a ceiling hoist:



7. Independently: The user grips the bed rail and turns to side lying position using the sliding properties of the Master Turner – S under the pelvis. A Positioning Wedge – S is recommended in front of the stomach for optimal support, security, and comfort.



8. Turning with one helper: The user grips the bed rail and begins the turn. The helper grabs the Master Turner – S and supports the turn to side lying position. Cushions and Wedges can be used for optimal support, security, and comfort.



9. Turning with a ceiling hoist: Connect the hoist to the handles. The user helps as best as he can while the hoist is elevated and supports the turning. Cushions and Wedges can be used for optimal support, security, and comfort.

User Manual - Master Turner - S

Prevent sliding down in bed:



10. User/helper positions the user's pelvis correct in bed using the sliding properties of the Master Turner. The position can be checked by fully elevating the knee rest.



11. Place two Positioning Wedges – XS, MC 001-8040, under the Master Turner – S under the user's thighs, with the anti-skid side downwards. Place a Positioning Cushion between the feet and the footboard with approximately 5 cm space.



12. Elevate the back rest. To create more stability, security and more body perception and awareness a Positioning Wedge - M, MC 001-8050, can be placed along each side of the upper body.

User Manual - Master Turner - S

Technical Specifications

Materials:

Master Turner Classic and Terry Cloth: Cotton, polyester, polyamide, nylon.

Master Turner Classic Inco - S: Cotton, polyester, polyamide, polyurethane, nylon.

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Handles:	Max. User Weight:
MC 001-1712 Master Turner Classic - S	L 70 x W 130 cm	4	150 kg
MC 001-2517 Master Turner Classic Inco - S	L 70 x W 130 cm	4	150 kg
MC 001-1912 Master Turner Terry Cloth - S	L 70 x W 130 cm	4	150 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Never leave the user lying on the product whilst connected to a hoist.



Do not use as a sling sheet. Can be combined with a hoist for e.g. repositioning in bed.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 150 kg



Washing:

This product can be washed in a washing machine at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk



Find us on:

