

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

14\_20230626\_UK

## Travel Master

The perfect turn system when traveling



Travel Master



	Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/>	MC 002-1328	69437	Travel Master Top	5744001971987
<input type="radio"/>	MC 002-1331	50301	Travel Master Bottom	5744001971994

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 20

The Original Nordic Human Handling Concept



Developed & Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

## **MDR (EU) 2017/745:**

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

Light and mobile turn system to reduce friction under the pelvis when turning and/or transferring to sitting on the edge of the bed. For users with reduced physical function.

Only with anti-skid on one side. Therefore, it is important to fit it correctly in the bed to avoid sliding off.

## **Proper use of the product:**

Place the two parts of Travel Master in the bed. The Top and the Bottom slide against each other during transferring and turning in bed. Travel Master is light and mobile which is why it is ideal, for example when travelling, as it is easy to pack and only takes up a small amount of space.

## **Warning:**

Always place the Travel Master Bottom with the grey anti-skid on the side to which you get out of bed. This to avoid sliding off and onto the floor.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the

user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, load and how often it is washed/dried. Products without nylon sliding material have a much longer life span.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - Travel Master

## Placement of Travel Master:



1. Travel Master Bottom should be placed in the center of the bed, with the velour facing downwards and the anti-skid edge towards the side where the user gets out of bed.



2. Travel Master Top is placed across onto of the bottom with the purple side upwards. Now it is ready for the user.



3. Alternatively place the Travel Master Top around the user with the label "This side up" towards the back.

## Getting into bed with the Travel Master:



4. Sit comfortably with the Travel Master Top on the edge of the bed where the Travel Master Bottom has the anti-skid edge.



5. The user turns easily into the bed as the friction is reduced by the two nylon surfaces. Elevate the knee-rest and the back rest if possible.



6. Straighten out the Travel Master if it is folded.

## Turning and getting out of bed with the Travel Master:



7. Carry out the turning or the adjustment of the position by pushing with the feet onto the sheet. If necessary, support the turning by using the bedrail.



8. Getting out of bed the user can fold the Travel Master around the waist. Elevate the knee rest and the backrest if possible and turn to side lying position.



9. The anti-skid edge of Travel Master protects the user from sliding of the edge of the bed.

# User Manual - Travel Master

## Technical Specifications

### Materials:

Cotton, polyester, polyamide, nylon.

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Handles:	Max. User Weight:
MC 002-1328 Travel Master Top	L 70 x W 90 cm	2	200 kg
MC 002-1331 Travel Master Bund	L 40 x W 85 cm	-	200 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Never leave the user lying on the product whilst connected to a hoist.



Do not use as a sling sheet. Can be combined with a hoist for e.g. repositioning in bed.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



### User Weight:

Recommended max. user weight: 150 kg



### Washing:

Travel Master Top can be machine washed at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



### Washing:

Travel Master Bottom can be machine washed at max. 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

Find us on:



For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)

