

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

15_20221229_UK


Master Belt



MC 001-1310



MC 001-1311

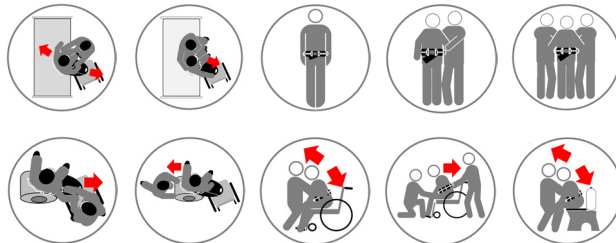

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 001-1310	109640	Basic Belt - M	5744001970393
<input type="radio"/> MC 001-1311	109943	Basic Belt - L	5744001973165

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 15 04

The Original Nordic Human Handling Concept



Developed & Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is used while facilitating walk or during sitting transfers with users who are physically and/ or mentally disabled.

The product is a facilitating aid and should not be used as a lifting aid as this can cause injuries to user and/or helper.

Correct use of the product:

Basic Belt is mounted around the user's waist. With the handles the helper can facilitate the user in different transfers without directly touching the user. The handles on the Basic Belt gives the helper the opportunity to get a more upright standing-position and therefore a better working posture.

Warning:

The Basic Belt is not intended as lifting-belt of any kind. Nor is the Basic Belt intended for use in combination with a hoist.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 2-5 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Basic Belt

Table of Contents

Fitting the Basic Belt:	page 2
Sitting to standing position:	page 2
Walking training with Basic Belt:	page 2
Sitting the user down from a standing position:	page 3
Transfer between wheelchair and bed:	page 3
Technical Specifications	page 4



User Manual - Basic Belt

Fitting the Basic Belt:



1. The Basic Belt is mounted around the waist of the user. Basic Belt is fastened using its adjustable buckle.



2. To customize the Basic Belt to the user's waistline, pull the end of the belt itself and in the opposite direction of the strap for tightening.

Sitting to standing position:



3. The user and the helper sit on the bed frame. The helper supports the user on the hips.



4. The helper supports the user whilst standing up.



5. A Rollator can be placed in front of the user.

Walking training with Basic Belt:



6. Helper 1 stands beside the user whilst guiding the user by using the handles on the Basic Belt.



7. If there are 2 helpers, the 2. helper can support the user's trunk or guide the legs.

User Manual - Basic Belt

Sitting the user down from a standing position:



8. The helper stands in front of the user and grips the handles on both sides of the Basic Belt.



9. The helper supports his/her knees against the user's knee whilst bending their knees with a straight back matching the user's movement as the user sits down in the bed/chair.

Transfer between wheelchair and bed:



10. The user has a Basic Belt fitted around the waist, and a Master Glide sliding board is placed between the wheelchair and bed.



11. The helper grasps the handles on both sides of the Basic Belt and supports the transfer to the side.

User Manual - Basic Belt

Technical Specifications

Materials:

Cotton, Alvelen, Nylon, Polyester

Article no. - Product Name	Size:	Number of handles:	Max. User Weight:
MC 001-1310 Basic Belt - M	Waist size: 75 - 110 cm	9	150 kg
MC 001-1311 Basic Belt - L	Waist size: 85 - 120 cm	11	150 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 150 kg



Washing:

This product can be washed in a washing machine at max 40 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. When damaged the product must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

Find us on:

