

# MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

# (UK) User Manual

16\_20231013\_UK

## Silk Master



Silk Master Pro - S



Silk Master Inco - M



Silk Master - L



Silk Master -XL



	Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/>	MC 001-1839	42491	Silk Master Inco - S	5744001973813
<input type="radio"/>	MC 001-1840	42493	Silk Master Inco - M	5744001970737
<input type="radio"/>	MC 001-1845	42494	Silk Master Inco - L	5744001973233
<input type="radio"/>	MC 001-1939	39116	Silk Master - S	5744001973271
<input type="radio"/>	MC 001-1940	33244	Silk Master - M	5744001970867
<input type="radio"/>	MC 001-1946	38934	Silk Master - XL	5744001970881
<input type="radio"/>	MC 002-1939	81089	Silk Master Pro - S	5744001972120
<input type="radio"/>	MC 002-1940	80335	Silk Master Pro - M	5744001972137
<input type="radio"/>	MC 002-1945	81090	Silk Master Pro - L	5744001972144
<input type="radio"/>	MC 002-1946	81091	Silk Master Pro - XL	5744001972151

Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic  
Human Handling Concept



Developed &  
Manufactured by:



# Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

## **MDR (EU) 2017/745:**

Products in this user manual are CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

## **The intended use of the product/-s:**

The intended use is to reduce friction against the surface while turning in the bed. Suitable for users with pain and reduced movement in hip, pelvic and/or back.

## **Correct use of the product:**

Silk Sheets can be used if the user has decreased function when repositioning or resting in bed. The Silk surface will reduce friction thereby making it easier to rotate in bed.

## **Risk assessment:**

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for trans-

ferring or moving a user, is ergonomic and legally sound.

## **Knowledge of the product/transferring:**

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

## **Life span:**

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 2-5 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Make visual inspections frequently and preferably before every use. Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

## **Guarantee:**

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

## **Unintended events and injuries when misusing the products:**

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



# User Manual - Silk Master

## Fitting the Silk Master:



1. Place Silk Master so that the silk part is positioned in the middle of the mattress and both sides are hanging over the edges of the bed.



2. Then, fold both sides under the mattress, making sure there are no folds on the silk part.

The Silk Masters with hook-and-loop tape can be mounted by bending the mattress and closing the hook-and-loop band on the backside. Alternatively the mattress can be raised on to one side and the hook-and-loop tape can be attached.

## Turning in bed:



3. The user bends the leg and grips the bed rail on the opposite side.



4. The user can easily slide to their side by pulling with their hand and lowering the leg onto the bed.

## To sitting position on the edge of the bed:



5. The user activates the bed's knee bending function.



6. The user then elevates the bed's headboard.



7. Due to the smooth silk layer, the user can now easily rotate on the side and slide their legs over the edge of the bed.

# User Manual - Silk Master

## Technical Specifications

### Materials:

Cotton, Polyester

### ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name	Size:	Sliding piece:	Max. User Weight:
MC 001-1839 Silk Master Inco - L	L 70 x W 200 cm	L 70 x W 60 cm	160 kg
MC 001-1840 Silk Master Inco - M	L 100 x W 200 cm	L 100 x W 60 cm	160 kg
MC 001-1845 Silk Master Inco - L	L 150 x W 200 cm	L 150 x W 60 cm	160 kg
MC 001-1939 Silk Master - S	L 70 x W 200 cm	L 70 X W 60 cm	160 kg
MC 001-1940 Silk Master - M	L 100 x W 200 cm	L 100 x W 60 cm	160 kg
MC 001-1946 Silk Master - XL	L 200 x W 200 cm	L 200 x W 60 cm	160 kg
MC 002-1939 Silk Master Pro - S	L 70 x W 200 cm	L 70 x W 60 cm	160 kg
MC 002-1940 Silk Master Pro - M	L 100 x W 200 cm	L 100 x W 60 cm	160 kg
MC 002-1945 Silk Master Pro - L	L 150 x W 200 cm	L 150 x W 60 cm	160 kg
MC 002-1946 Silk Master Pro - XL	L 200 x W 200 cm	L 200 x W 60 cm	160 kg

## WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Make sure that all relevant brakes are activated before transferring the user.



### CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

For more information, videos Quick Guide etc., visit [www.master-care.dk](http://www.master-care.dk)



### User Weight:

Recommended max. user weight: 160 kg.



### Washing:

Silk Master can be washed in a washing machine at max. 60 °C.

Silk Master Pro and Satin at max. 85 °C.

Gentle washing and dry tumbling significantly prolongs the durability.



### Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



### Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



### Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C.

Airdrying prolongs the durability significantly.



### Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



### Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



### Ironing:

Do not iron.



### Dry cleaning:

Do not dry clean.



### Bleach:

Do not bleach.



### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



### User Manual:

Always read the User Manual thoroughly before using the product.



### Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

Find us on:

