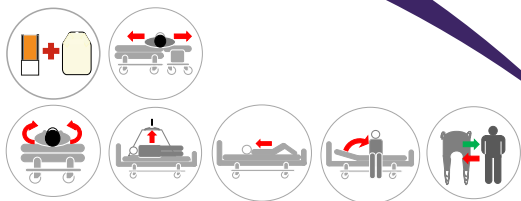
# MADE IN EUROPE

This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

**(DK)** Forordning MDR 2017/745 for medicinsk udstyr klasse 1  
**(UK)** Regulation MDR 2017/745 for medical devices class 1

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### Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.

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**(UK)** Never leave the user on the product when lying on the edge of the bed.

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**(UK)** Never leave the product on the floor.

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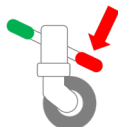
**(UK)** Never leave the user lying on the product whilst connected to a hoist.

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**(UK)** Always make sure that the user is not caught in the straps.

**(DK)** Sørg for, at alle relevante bremses er låst, inden forflytning foretages.

**(UK)** Make sure that all relevant brakes are activated before transferring the user.

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**(UK)** Do not use as a sling – can be combined with a hoist for repositioning in bed.

ISO 12 31 03 21

The Original Nordic  
Human Handling Concept


**(DK)** For mere information, brugervejledning, videoer mm., se  
**(UK)** For more information, User Manual, videos etc. visit

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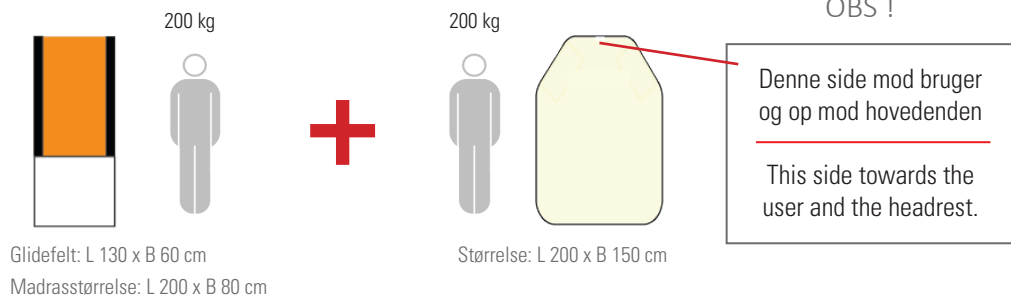### 1. Turning to sidelying (manually):

Bend the user's knees. One helper places their hands on the user's hip and shoulder thus stabilizing the user while the other helper pulls the Turn Sheet turning the user to sidelying.

### 2. Turning with hoist:

Bend the user's knees. Mount the Turn Sheet on the hoist and elevate slightly. Push the user backwards. Elevate the hoist if necessary and push the user further backwards. If needed use a Master Comfort Positioning Wedges - S to increase the sense of stability for the user.

### 3. Higher up in bed:

Manually: Elevate the foot end. The helper's hands are placed on the user's ankles while the user pushes the body towards the top of the bed. With a hoist: Mount the lift on the Turn Sheet around the user's hip and elevate slightly. Pull the user upwards from the side or the top of the bed.

**OBS !! Vendelagnet KAN IKKE VASKES / ATTENTION !! The Turn Sheet CAN NOT be washed**

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(UK) Gentle washing and dry tumbling significantly prolongs the durability.

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(UK) Never use fabric softener as this will reduce the sliding properties significantly.

Find us on:

master care A/S, Sofienlystvej 3, DK 8340 Malling, Danmark  
Tel. +45 86 93 85 85, E-mail: info@master-care.dk