

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.


(UK) User Manual

22_20240619_UK

Dress Master

Pat. BA 2022 00046

Gebrauchsmusters Nr. 20 2023 105 598

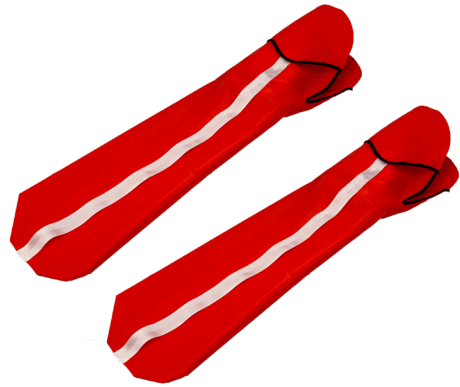

Article no.	HMI no.	Name - Size	UDI-DI/GTIN
MC 001-0995	133644	Dress Master	5744001974100

Regulation MDR 2017/745 for medical devices class 1



ISO 12 03 03 20

The Original Nordic Human Handling Concept



Developed & Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S or our distributor in your area.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is a sliding tube for dressing. The double sliding aspect reduces the risk of shear injuries and reduces the ergonomic load for the helper during dressing.

Correct use of the product:

Slide the Dress Master on the user's leg(-s) /arm(-s) before sliding the cloth over the Dress Master. When the cloth is on remove the Dress Master by pulling lightly in the bands.

Contraindications:

Do not use the product if the patient has open wounds. Always dress wounds before using the Dress Master.

Warning:

Never leave the Dress Master on the floor due to risk of falling.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for dressing the user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S and our distributors provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of products with nylon sliding material with daily wear and tear and washing averagely is 1-3 years. The products life span depends on the use, users load, washing and drying temperatures and how often it is washed and dried.

Products with insufficient friction reduction, defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the appropriate National Authority and to master care A/S or the distributor in the current country.

master care A/S and our distributors take no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



User Manual - Dress Master

How to use the Dress Master:



1. A large sliding piece can be placed under the user's legs and hips to minimize friction against the sheet.



2. Place the Dress Master with the short side near the crotch, the long side outwards, and the pull straps on the outside.



3. Slide the Dress Master over the foot and up along the leg. Make sure to place the sliding material all the way up under the user's pelvis.



4. If the user cannot use either leg apply the Dress Master on both legs.



5. Maintain the Dress Master between the user's legs and the trousers while pulling the trousers up beneath the users pelvis.



6. Make sure the trousers are pulled all the way up and straightened under the user before closing them.



7. Remove the Dress Masters one at a time by pulling the two straps simultaneously.



8. Remove the sliding material under the user after removing the Dress Masters.



9. The Dress Master can also be used for shirts, jackets, and cardigans.

User Manual - Dress Master

Technical Specifications

ISO certifications:

ISO 12952-1:2015

Textiles – Assessing the ignitability of all bedding items, when subject to a smouldering cigarette.

Article no. - Product Name:	Size:	Max. user height:	Max. user weight:	Product contents:	Product weight:
001-0995 Dress Master	L 150 x W 37/29 cm	Not relevant	150 kg	97% Polyester, 76% Polyamide, 8% Silicone.	0,15 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



Always make sure that the user is not caught in the straps.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.



User Weight:

Recommended max. user weight: 150 kg



Washing:

This product can be washed in a washing machine at max 85 °C. Gentle washing and dry tumbling significantly prolongs the durability.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

This product may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

This product tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make visual inspections frequently and preferably before every use. Pay attention to torn fabric, seams and straps. Products with high friction or other deficiencies must be discarded.



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

For more information, videos Quick Guide etc., visit www.master-care.dk

Find us on:

