

MADE IN EUROPE



This product has been developed and manufactured using both raw materials and with production in Europe.

Choose quality, save money, and protect the environment.

(UK) User Manual

38_20231013_UK

Master Comfort Sleep

The perfect cushion for sleeping



Master Comfort Sleep



Regulation MDR 2017/745 for medical devices class 1



ISO 12 31 03 21

The Original Nordic Human Handling Concept



Article no.	HMI no.	Name - Size	UDI-DI/GTIN
<input type="radio"/> MC 001-8085	136598	Master Comfort Sleep	5744001974650
<input type="radio"/> MC 001-8086	136637	Master Comfort Sleep - Cover	5744001974667

Developed & Manufactured by:



Thank you for choosing a product from master care A/S.

We recommend reading the user manual thoroughly before using the product (-s).

The information in this user manual applies to the shown products provided that the products have not been modified or customized and that the products have been used as recommended in this user manual.

If the customer changes, repairs or uses the product in other combinations than recommended by master care A/S the CE-approval no longer applies and the guarantee becomes void.

In case of doubt, contact master care A/S.

MDR (EU) 2017/745:

Products in this user manual is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation MDR (EU) 2017/745 – Class 1.

The intended use of the product/-s:

The product is used to support and reduce pressure on the tissue while repositioning a user or when lying in bed. Wedges with anti-slide on the underside can be used directly on slippery surfaces.

Proper use of the product:

The Master Comfort products can be placed between the user and the area of support and thereby reduce the pressure on the skin. Alternatively, the products can be used to support the sitting position or when lying on the side to prevent falling forward or backwards.

Warning:

The use of pillows and wedges can't replace the need of frequent repositioning and daily skin assessment.

Risk assessment:

Always make a risk assessment and a functional assessment of the user prior to using the product. This way using the product for transferring or moving a user, is ergonomic and legally sound.

Knowledge of the product/transferring:

It is important that the user of the product has been properly instructed in the use of the product and has read the user manual thoroughly prior to using the product.

master care A/S provide education in the use of products and on our website all user manuals can be found.

Life span:

The expected life span of the products is 1 - 5 years depending on the use, user's load and how often it is washed and dried.

Products which no longer provide sufficient support, with defect seams, broken material or other deficiencies must be discarded. This way you will always keep a sound and healthy work environment for both helper and user.

Make visual inspections frequently and preferably before every use.

Guarantee:

master care A/S provide a 2-year guarantee covering material defects and manufacturing defects if the product has been used correctly.

Unintended events and injuries when misusing the products:

Unintended events must always be reported to the correct national authority and to master care A/S or the distributor.

master care A/S takes no responsibility for personal injuries or product damages if the product(-s) have been used in contravention of the by the manufacturer recommended intended purpose or proper use of the product.



master-care.dk

User Manual - Master Comfort Sleep

Placing the Master Comfort Sleep:



1. Master Comfort Sleep provides a stabile positioning in bed. Can be placed either in front of the body or behind the back and legs. The top can be used as a pillow.



2. Place the Master Comfort Sleep behind the user's head, back and under the upper leg.



3. The lower part of the Master Comfort Sleep can be modulated to give optimal support to the upper leg or be placed between the legs to provide pressure relief.

Seated positioning:



4. Master Comfort can be placed around the body to give increased stability and body awareness.



5. Place the middle of Master Comfort Sleep under the head. Leave the Bed Rail up to keep the Krøyer balls in place.



6. Place and adjust the left and right side of Master Comfort Sleep along the body. Adjust the balls to comfortably support the forearms and hands.

Cover:



7. The Master Comfort Sleep is delivered with a incontinence surface. The fabric is stretchable and has a long durability. It can be cleaned with soap and water or with alcohol.



8. A soft, comfortable and washable cover is available to protect the cushion.



9. Master Comfort Sleep consists of real Krøyer balls which adadded to the body and give the body a sense of stability and a good pressure relief.

User Manual - Master Comfort Sleep

Technical Specifications

Materials:

Cover: Flame retardant, elastic, and waterproof polyester.

Extra Cover: Cotton, Elastane

Contents: Original, flame retardant Krøyer balls (expandable polystyrene).

ISO certifications:

DS/EN ISO 12952-1:2010

Textiles – Assessment of the ignitability of bedding items – Part 1: Ignition source: smouldering cigarette.

DS/EN ISO 12952-2:2010

Textiles – Assessment of the ignitability of bedding items – Part 2: Ignition source: match-flame equivalent.

Article no. - Product Name:	Size:	Max. User Weight:
MC 001-8085 Master Comfort Sleep	L 200 x W 50 cm	200 kg
MC 001-8086 Master Comfort Sleep - Cover	L 200 x W 50 cm	200 kg

WARNINGS



Never leave the user on the product when lying on the edge of the bed.



Never leave the product on the floor.



The use of pillows and wedges can't replace the need of frequent repositioning and daily skin assessment.



Make sure that all relevant brakes are activated before transferring the user.



CE-certification:

This product is CE-approved and follows the harmonized standards. This means that the products are in compliance with the essential health and safety directives. The product(-s) comply with the standards in the Medical Device Regulation (EU) 2017/745 – class 1.

For more information, videos Quick Guide etc., visit www.master-care.dk



User Weight:

Recommended max. user weight: 200 kg.



Washing:

The Cover can be washed in a washing machine at max. 85 °C. Lower washing temperature will prolong the durability. The Master Comfort Sleep can be disinfection with alcohol.



Detergents pH level:

This product tolerates detergents / disinfectants with a pH level between 5-9.



Fabric softener:

Never use fabric softener as this will reduce the sliding properties significantly.



Dry Tumbling:

The Cover may only be dry tumbled gently and at max. 40 °C. Airdrying prolongs the durability significantly.



Surface cleaning:

The surface of this product can be cleaned with a wet cloth and soap.



Disinfectants:

The Master Comfort Sleep tolerates alcoholic disinfectants with a pH level between 5-9.



Ironing:

Do not iron.



Dry cleaning:

Do not dry clean.



Bleach:

Do not bleach.



Maintenance:

Make frequent visual inspections to check that the incontinence surface and all seams are intact before use



User Manual:

Always read the User Manual thoroughly before using the product.



Disposal:

Always make sure to follow local guidelines when disposing or discarding the product.

Find us on:

